To fit the mouth tray to the upper teeth:

- Put out two bowls with enough capacity to submerge the whole mouth tray:
 - 1 bowl with boiling water
 - 1 bowl with cold water
- Hold the mouth tray by the flexible end and immerse it entirely in boiling water for 15 seconds (do not exceed this time because it could lose its shape entirely).
- Remove the mouth tray from the boiling water. Shake off all the water vigorously.
- Immediately place it into your upper mouth. It feels hot but should not burn. Carefully place your teeth and molars firmly against one another and press your tongue against the back of your front teeth. In this position, suck off any water and air for a few seconds (in so doing, you improve the contact with your teeth). Massage your lips and teeth with your fingers so that the mouth tray fits your teeth as close as possible.
- After 1 minute, remove the mouth tray from your mouth and immerse immediately in cold water. After at least 3 minutes, take the mouth tray out of the cold water. It is now ready for use.

Fit the second mouth tray to the lower teeth:

Use the second tray and follow the same process as with the upper teeth, but now with the lower mouth.

INSTRUCTIONS FOR THE USE OF THE GEL IN THE MOUTH TRAYS

- Before you start to use the whitening gel, clean and brush your teeth preferably with Safe & white® toothpaste for at least 2 minutes (upper and lower).
- The mouth trays must be dry and clean.
- Apply Safe & white® whitening gel in the tray where the molars and teeth are to be lightened. Look at the tray carefully so that you know where the gel has to be applied.

- The whitening gel must be applied thinly to the required place in the tray, then place it in your mouth.
- Press the tray well onto the teeth and leave it in your mouth for at least 30 minutes. During this time you must not eat or drink anything. Excess saliva mixed with the Whitening gel must be spat out.
- After removing the tray, rinse your teeth and mouth well with water. Rinse the tray clean and allow it to dry thoroughly.

The treatment may be repeated until the desired result is obtained.

In most cases the process will take 2 weeks. If some teeth remain darker than others (canines for instance), you can lighten them in the following way: apply just a little whitening gel to the tray at the site of the darker tooth (or teeth). Do not apply any whitening gel at the site of teeth/molars that are already whitened sufficiently. When Safe & white® whitening gel is used as described above, the first results are visible after one week. We advise you to use Safe & white® whitening gel twice a day for 30 minutes for a minimum of two weeks. Consecutive treatment sessions can accelerate the whitening process.

DISCONTINUATION OF TREATMENT

Treatment can be discontinued for one or more days. The best result, however, is achieved with a continuous treatment.

MEASURE THE WHITENING RESULTS WITH THE SHADE GUIDE

Use the shade guide in front of the mirror to measure the whiteness of your teeth and smile before and after using the Safe & white® whitening gel. Depending on how effective you apply the gel in conjunction with the instructions you can achieve up to 7 shades whiter teeth after two weeks. The final result will always depend on the cause of the

discolouration. Brown and yellow teeth (mainly the result of tobacco, coffee, tea and wine) become on average 4 shades lighter. Grey teeth (caused by discolouration of dentine within the tooth as a result of medication) will become average 2 to 3 shades lighter.

PRECAUTIONS

- Keep out of the reach of children
- Use the Safe & white® whitening gel and the tray only as described above.
- Avoid contact with the eyes. In the event of eye contact, rinse thoroughly with water.
- Store in a cool, dry place away from heat and direct sunlight (8-21°Celsius)
- Do not use when pregnant

SAFE & WHITE® WHITENING GEL HAS BEEN TESTED BY

- TUFTS University, Boston, USA
- Loma Linda University, California, USA
- University of Indiana, USA
- St. Bartholomew's and Royal London School of Medicine, UK
- Dental Academic Centre Amsterdam, The Netherlands
- Mubio Research BV, University of Maastricht, The Netherlands

CONTENTS

- 100ml Safe & white® whitening gel
- Instructions
- 2 mouth trays which can be easily adjusted to fit your mouth
- Shade guide

For more information www.vitint.com

Manufactured by Ardoz Research BV, 's-Hertogenbosch, The Netherlands

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Safe & white® whitening gel is a whitening product for your teeth. It is a "Home Whitening System", with application through a "mouth tray". The action of Safe & white® whitening gel is based on Ardox-X®, which does not contain peroxide, nor does the formation of free radicals occur.

SAFE

Safe & white® whitening gel can be used without risk by patients with gum problems. In addition, Safe & white® whitening gel has no acidic pH. As a result, it is absolutely safe for dental enamel and gums. This innovative product also has the following advantages:

- Harmless to dental enamel, as demonstrated by Electronic Microscope photos.
- No gum and teeth sensitiveness.
- No impairment of bonding to dental enamel. Products containing peroxides cause damage that weakens bonding. It is therefore advised that fillings should not be performed for several days after whitening. With Safe & white® whitening gel, however, a filling can be performed immediately after whitening.
- Result remains visible longer. Products containing peroxides cause damage to the enamel and as a result stains bind more rapidly to the enamel after whitening with peroxide.

INSTRUCTIONS FOR USE SAFE & WHITE® WHITENING KIT

For long lasting natural, white teeth.

Hence, teeth discolour more rapidly. Safe & white® whitening gel does not damage the enamel and therefore teeth remain whiter much longer.

ARDOX-X® TECHNOLOGIE

Ardox-X® is a oxygen donating complex. It produces a whitening effect through an oxidative process. The release of oxygen is stabilized at the cell level within the complex. Unlike peroxides, no harmful free radicals are formed during the oxidation process so therefore it is completely harmless to the enamel and gum.

SAFE & WHITE® WHITENING GEL

Teeth are gradually whitened with Safe & white® whitening gel. Safe & white® whitening gel dissolves in water and in saliva. The whitening gel is non-abrasive and causes no damage or sensitiveness to teeth and gum.

HOW TO USE SAFE & WHITE® WHITENING GEL

- Use of mouth trays
- Application of the gel

INSTRUCTIONS FOR THE USE OF THE MOUTH TRAYS

In the pack you will find two trays: one for the upper and one for the lower elements.